



FLOWABLE COMPOSITE

For Multiple Indications



Use within 12 months
of opening the box



Keep dry



See instruction
for use



Keep Protected
from sunlight



Non sterile

Rx Only

Before Use Please Read The Following Instructions Carefully

Product For Dental Use Only

CAUTION:

Flowable Composite is a medium viscosity light cure Nano filled resin restorative suitable for Class II (base/liner), Class III and class V restorations. This Nano composite technology incorporates three different fillers resulting in a flowable, yet sculpatble and non-slumping material. Additional indications include: Pit and fissure sealant, repair of enamel defects, repair of porcelain restorations, minor occlusal build-ups in non-stress bearing areas and incisal abrasions.

Ingredients:

Methacrylic resins (bis-GMA, UDMA ,TGDMA), inorganic nanoparticle filling, pigments, photoinitiator, inhibitor.

Indications for use:

Composite Fillings, Cavity Liner, Pit and Fissure Sealant, Porcelain Repair, Margin Repair, Small Core Build-Ups, Buccal Repair, Porcelain Veneer Cementation, Bonding Splints, Class III, Class V and Small Class IV's.

Instructions for use:

Composite Filling

- Isolate and prepare the tooth surface to be restored.
- Etch the surface with ETCH 37%. Wait for 20 seconds. Rinse with water for 20 seconds.
- Apply an adhesive, such as the 5th GEN Universal Adhesive System or 7th GEN Self-Etch Adhesive System, according to the manufacturer's instructions.
- Using the appropriate shade of Flowable Composite, place a 1mm to 2mm increment of Flowable Composite into the cavity preparation along the axial and gingival margins. Use a brushing action to allow the material to flow into the marginal interfaces and recesses of the preparation. Light-activate for 20 seconds.
- Continue to build the restoration incrementally until the cavity preparation is filled to the desired level. Flow the composite into the preparation with a brush like motion. Light-cure each increment for 20 seconds
- Shape and finish to desired level.
- Retract the composite syringe piston and seal the container with the original cap immediately after use to avoid polymerization of the product inside.
- The product is intended for repeated use until spent. The supplied applicator is disposable. Reusing the applicator may cause a risk of infection.

Contraindications:

Do not use in patients who are hypersensitive to any of the ingredients.

Precautions:

- Refrigerate Flowable Composite when not in use.
- This product can cause eye irritation in the uncured form. In case of contact, rinse with plenty of water and contact a doctor immediately.
- If swallowed, drink plenty of water. Do not induce vomiting. Contact a doctor immediately.
- Use rubber dams, disposable gloves, safety goggles and protective clothing when working with this product.

Storage:

- Store in a tightly closed original container below 25 °C.
- Protect from direct sunlight. Reseal after use.
- Keep out of the reach of children.
- The expiry date is on the direct packaging.
- Opening the originally sealed container does not change the expiry date if the product is resealed after each use.

Disposal of packaging:

Waste containers should be disposed of or returned to the manufacturer.

Shelf Life: 2 years from the date of manufacturing.

Packaging:

Syringe containing 2 gm. of Flowable Composite with disposable applicators.